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#### FORENSIC SWAB

Swabs for Forensic use, suitable for sample collection for DNA evidence analysis (data bank, paternity tests, etc.) and for investigation at crime scenes. They are certified Human DNA, DNase, RNase and PCR inhibitors free. It is available, for each lot of manufactured swab, a certificate of analysis issued by an accreditate laboratory Accredia ISO/IEC 17025. Swabs with plastic stick and rayon tip in test tube Ø 12x150 mm, with label for a correct and unambiguous sample identification, that acts as well as a security seal before the opening. ETO sterilized. Atoxic, not pyrogenic, Latex-free and phthalate-free device.

CARATTERISTICHE PRINCIPALI	TECHNICAL FEATURES	
Stato microbiologico	<b>STERILE / STERILE</b>	Microbiological status
Materiale asta	POLISTIROLO ANTI-SHOCK POLYSTYRENE ANTI-SHOCK	Raw material – applicator stick
Materiale puntale in fibra	VISCOSA / RAYON	Raw material – fibre tip
Materiale provetta	POLIPROPILENE / POLYPROPYLENE	Raw material – test tube
Materiale tappo	POLIETILENE / POLYETHYLENE	Raw material – cap
Lunghezza totale (mm)	145 ±2	Total length (mm)
Dimensioni asta (mm)	Ø 2,5 ±0,1 x 130 ±2	Applicator stick dimensions (mm)
Dimensioni puntale in fibra (mm)	Ø 4,5 ±1 x 15 ±3	Fibre tip dimensions (mm)
Assorbenza	100 µL ±10% in 10 sec. (acqua + TSB) / (water + TSB)	Absorbency
Validità	5 ANNI / YEARS	Shelf life

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*Swabs are sterile and ready-for-use systems intended for samples drawing and transport for sample collection for DNA evidence analysis. The swab is suitable for short contact with the human body. **Not suitable for diagnostic purpose.** Nuova Aptaca is not responsible for any unauthorized use. The device has not CE marking (it is not a medical device) as it is not intended for diagnostic purposes.*

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*Each batch of swabs is tested by an external laboratory accredited Accredia ISO / IEC 17025 in order to verify the absence of human DNA and PCR inhibitors (and the presence of human DNA on the control swab). The test is performed with the Real Time PCR technique with a detection limit of <16 pg /µL.. DNA concentration detection range: from 0.023 ng/uL to > 50 ng/uL*

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*Each batch of swabs is tested by an external laboratory accredited in order to verify the absence of DNase & RNase. The test is performed with the fluorimetric technique (incubation temperature +37 °C ± 1 °C - Incubation time 1 hour).*

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*The transport tube is without ventilation. The following tests carried out for the preservation of human DNA on swabs contaminated with sample salivary known and stored at a temperature of + 20 °C. Detection limit of <16 pg /µL.. DNA concentration detection range: from 0.023 ng/uL to > 50 ng/uL*

	TEMPO DI CONSERVAZIONE / TIME				
	0 h.	12 h.	24 h.	48 h.	96 h.
<b>DNA conc. ng/ul</b>	4,78	4,74	4,16	3,55	2,93
<b>Presenza inibitori PCR inhibitors</b>	negativo not present	negativo not present	negativo not present	negativo not present	negativo not present

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- *Follow aseptic techniques when using the product.*

- *Each sample shall be assumed to be containing infectious micro-organisms and therefore it shall be treated with the necessary safety measures. After use, test tubes and swabs shall be disposed of according to the laboratory provisions concerning infectious material.*

- *Strictly follow the user's instructions.*

- *When using a device with plastic or wooden stick, the pressure applied during sampling shall be light since the stick material is breakable.*

- *The fibre adhesion to the stick is tested for instantaneous sampling; a longer contact between the swab and the sampling area might cause the fibre to come out.*

- *If the swab is submitted to a chemical or physical sterilizing or micro-biostatic process, its intended functioning could be compromised.*

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a) *Extract the swab and carry out sampling – During sampling, the swab end shall only come in contact with the area of sampling, so as to reduce contamination risks. WARNING: during sampling, do not apply an excessive force which could break the stick*

b) *Put the swab in the transport test tube*

c) *Write the sample data on the tube label*

d) *Send the sample to the laboratory for an immediate test.*

e) *The only device's purpose is for DNA analysis, not for diagnostic purpose*

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*The content of the unopened and undamaged units is guaranteed to be sterile. Do not use the units in case of damage, dehydration or contamination.*

*Do not use if the expiration date has passed.*

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*All raw materials, components and every lot of finished product undergo strict quality inspections carried out by N.A. in order to check conformance to specifications.*

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